

Automatic Shoe Cover Dispenser



CE RoHS

NEW!

Model: QY-II 200

- Voltage: 110V±10% 220V±10%
- Power: 20W
- N.W:23kgs
- G.W:27kgs
- Packing: 1pcs/carton

- SHOE COVER STORAGE BOX
- WARNING LIGHT
- STERILIZING LIGHT
- COUNTING MONITOR
- POWER SWITCH
- COUNT RIGHTING SWITCH
- ULTRAVIOLET RADIATION SWITCH

- STANCE
- HOUSING

● SANDBAG

● SHOE COVER BOX

Capacity PE Shoe Cover:- 200 pcs
Capacity CPE Shoe Cover:- 160 pcs

QY-II200

USING INSTRUCTION

1. Lightly arrange the shoe covers through the windows of the shoe cover box. even can expand the window follow the broken line which in the box. In order to make sure the center of the elastic in the middle of the exit of the box. Open the shoe cover box, take the splint of shoe covers out, put the sandbag in
2. Open the shoe cover storage box, Then insert the shoe cover box in the rabbit of machine, Make sure the cover is downwards.
3. Connecting power, push the power switch, the dispenser begin to work.
4. Put one foot into stance, step on.
5. Put another foot as last step.
6. After using, please turn on the UV light to disinfect. The light can turn off after disinfect. It spends 30 minutes.



MERINO INTERNATIONAL

Ahead... in medical technology innovations

B-40, (G.F) Pal Mohan Plaza, 11/56, Main Desh Bandhu Gupta Road,
Karol Bagh, New Delhi - 110 005 (INDIA)
Tel +91 11 455 88229, 2352 0809 Tele Fax +91 11 2352 0809 Cell +91 999 999 9615
E-mail merinointernational@gmail.com Web. www.merinointernational.com
An ISO 9001:2008 Accredited Company

